Barreira Gengival

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Gingival Barrier

USE INSTRUCTIONS

INDICATION: It is indicated to isolate and protect gingival tissues. It can be used for relative isolation in case of tooth whitening or as an adjunct to absolute isolation in restorative, rehabilitative procedures and/or endodontic treatment.

COMPOSITION: UDMA, HEMA, Camphorquinone, Tertiary amine: 4-ethyl dimethylaminobenzoate, Nanometric Silica and Pigments.

HOW TO USE: Confirm that the application tip is unobstructed. Dry the gum in the region where the isolation will be performed. Apply a thin layer of the gingival barrier over the marginal gingiva and interdental papillae. The gingival barrier should cover 0.5 to 1 mm of the neck of the adjacent tooth. After application, cure 20 seconds per area. At the end of the clinical procedure, remove the barrier by detaching it with an exploratory probe.

WARNINGS: The product should only be used on healthy gums. Incomplete or excessive polymerization may cause mild gingival irritation. Professional use. Product ready to use. Follow the

directions for use. Use PPE to handle the product. Avoid contact with eyes. In case of contact with the ocular mucosa, wash immediately with plenty of water.

CONTRAINDICATION: The product is contraindicated for patients who are allergic and/or sensitive to methacrylate monomer. The gingival barrier is not indicated for patients who have any gingival irritation and/or who have recently undergone oral surgery. Discontinue use in case of allergic reaction.

PRECAUTIONS: Keep out of the reach of children. Do not use the product after the expiration date. Do not use if the packaging is broken.

STORAGE: Keep the product in its packaging always closed. Protect from direct sunlight. Store the product at a temperature of 5 ° C to 28 ° C. Humidity up to 75%.

PRODUCT EXPIRY DATE: 24 months.

DISPOSAL: Follow the current legislation for the disposal of dental products.

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